

8000R

SpO₂ Forehead Sensor

Use **ONLY** with compatible Nonin Medical pulse oximeter monitors.

1. Remove adhesive backing from one horseshoe shaped adhesive ring.
2. Press the clear holder on the ring so a slot is open for the sensor cable. Remove from the card.
3. Press a clean 8000R sensor into the grey foam, with the emitter/detector facing **AWAY** from the foam, and cable in slot.
4. Wipe the forehead with an alcohol swab. Allow to dry.
5. Gently press on the forehead.
6. Apply a piece of medical tape to keep it firmly in place.



ADDITIONAL GUIDANCE

- A.** Check sensor position and skin at site every 6 to 8 hours.
- B.** If signal quality is low on the oximeter:
 - 1. Check connection at oximeter.
 - 2. Check that forehead is warm and has good blood flow.
 - 3. Use another location.
- C.** Clean sensor between patients or as needed:
 - 1. Remove and discard the used 8000H holder.
 - 2. Use soft cloth with 10% bleach/90% water solution (household bleach < 10% sodium hypochlorite) to wipe the sensor and cable.
 - 3. Do not use ammonium chloride-based cleaning fluids.
 - 4. Wipe away from connector plug. Do not submerge or immerse sensor or connector plug.
 - 5. Allow to completely dry before next use.

THIS GUIDE IS FOR REFERENCE ONLY. Refer to the Instructions for Use for complete information, warnings, cautions and specifications for the product you are using.

Visit nonin.com for more information and video tutorials.

R_X only



nonin.com/sensors

Nonin Medical, Inc.
13700 1st Avenue North
Plymouth, MN 55441-5443 • USA
Tel: +1.763.553.9968
+1.800.356.8874
Fax: +1.763.577.5521
E-mail: info@nonin.com

Nonin Medical B.V.
Prins Hendriklaan 26
1075 BD Amsterdam • Netherlands
Tel: +31 (0)13-79 99 040
Fax: +31 (0)13-79 99 042
E-mail: infointl@nonin.com