




Mounting Assembly Installation Instructions for Use (IFU) Nonin Medical Tabletop Pole Mount Assembly

This IFU describes how to attach the Tabletop Pole Mount Assembly.


Parts Reference

The Assembly includes the following parts and hardware (see photos below):

Item #	Description	Qty
1	C-Clamp	1
2	Mounting Adapter	1
3	#1/4-20 x 1/2" Knob Screw	1
4	#8-32 x 7/16" Flat Head Machine Screw (FHMS)	2

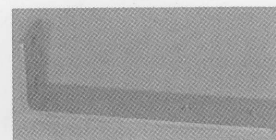

Mounting Adapter

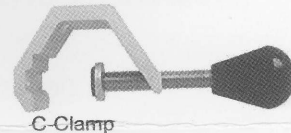

C-Clamp

Tools Required: Phillips screwdriver.




This accessory is intended for use with Nonin's Tabletop Monitors only.




The maximum recommended load for this accessory is five (5) pounds.

Attaching the Mounting Adapter to the Avant[®] Series Monitor

1. Place the monitor face down for access to the threaded mounting hole on the bottom housing (Picture 1).
2. Attach the Mounting Adapter to the bottom of the monitor using the #1/4-20 x 1/2" Knob Screw and the threaded hole closest to the right angle of the mounting adapter (Picture 2).

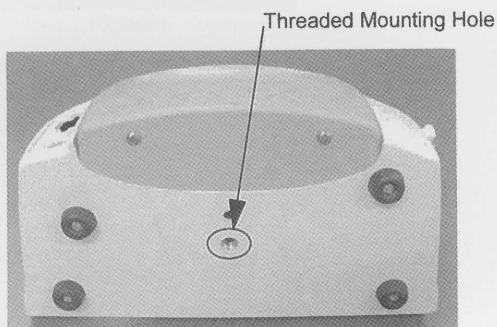


Picture 1

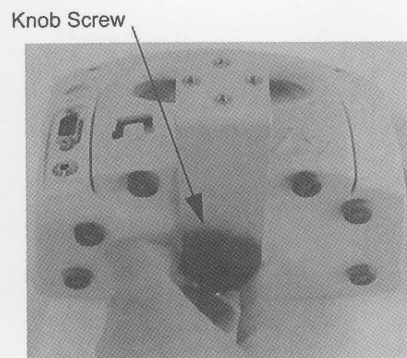


Picture 2

Attaching the Mounting Adapter to the Model 7500 Series Monitor

1. Place the monitor face down for access to the threaded mounting hole on the bottom housing (Picture 3).
2. Attach the Mounting Adapter to the bottom of the monitor using the #1/4-20 x 1/2" Knob Screw and the threaded hole farthest from the adapter bend (Picture 4).

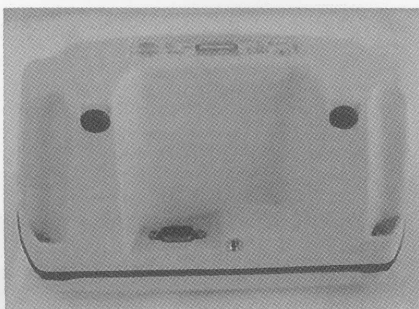

Picture 3

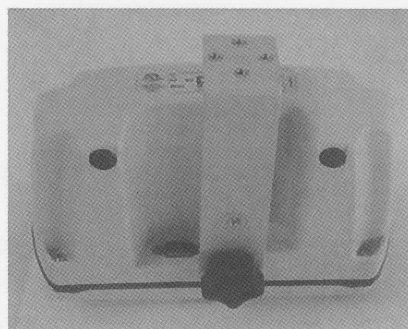


Picture 4


Attaching the C-Clamp to the Mounting Adapter for use with NONIN Tabletop Monitors

Installation Note: There are four (4) threaded holes in the Mounting Adapter (Picture 5) for either vertical or horizontal mounting of the C-Clamp. There are two (2) vertical and two (2) horizontal positions.


1. Align the C-Clamp in the desired mounting position (horizontal or vertical).
2. Attach the C-Clamp to the Mounting Adapter using two (2) #8-32 x 7/16" FHMS (Pictures 6 and 7).

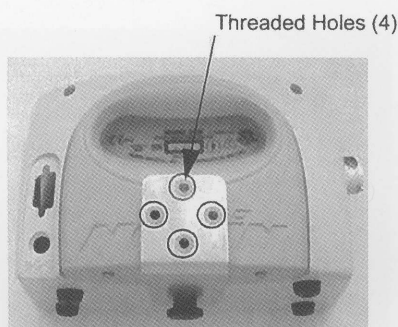

Picture 5

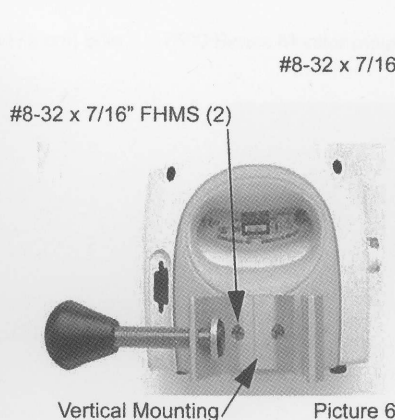

Picture 6

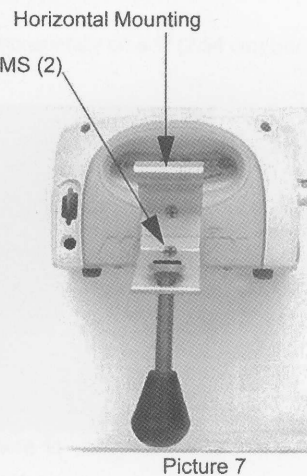

Picture 7


Mounting the Monitor

Installation Note: The C-Clamp will fit on a vertical pole or horizontal bar up to 1.75" (4.445 cm) in diameter.


1. Open the C-Clamp.
2. Slide it around the pole or bar.
3. Use the knob to tighten until the monitor is securely mounted.

Pictures 8 and 9 show examples of vertical and horizontal mountings (Pictures 10 and 11 for the 7500-Series).

Avant® Series Monitor mounted vertically on a 1.25" (3.175 cm) pole Avant® Series monitor mounted horizontally on a 1" (2.54 cm) bar

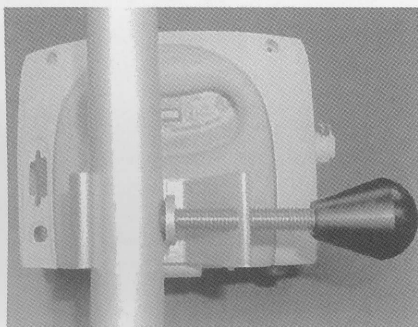


Picture 8

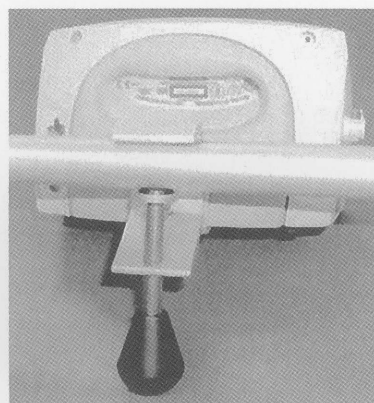


Picture 9

7500 Series Monitor mounted vertically on a 1.25" (3.175 cm) pole

7500 Series Monitor mounted horizontally on a 1" (2.54 cm) bar

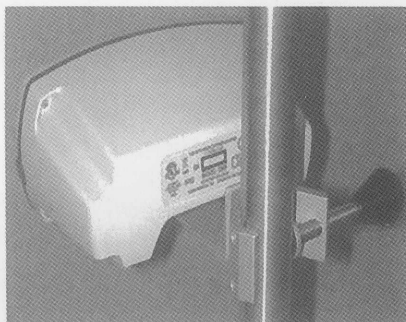

Picture 10

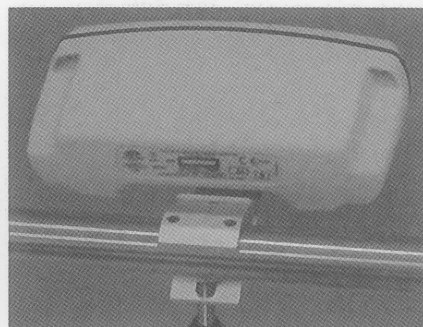

Picture 11